


Basket, Headlight, and Tiller Shroud Invacare 400 Scooter

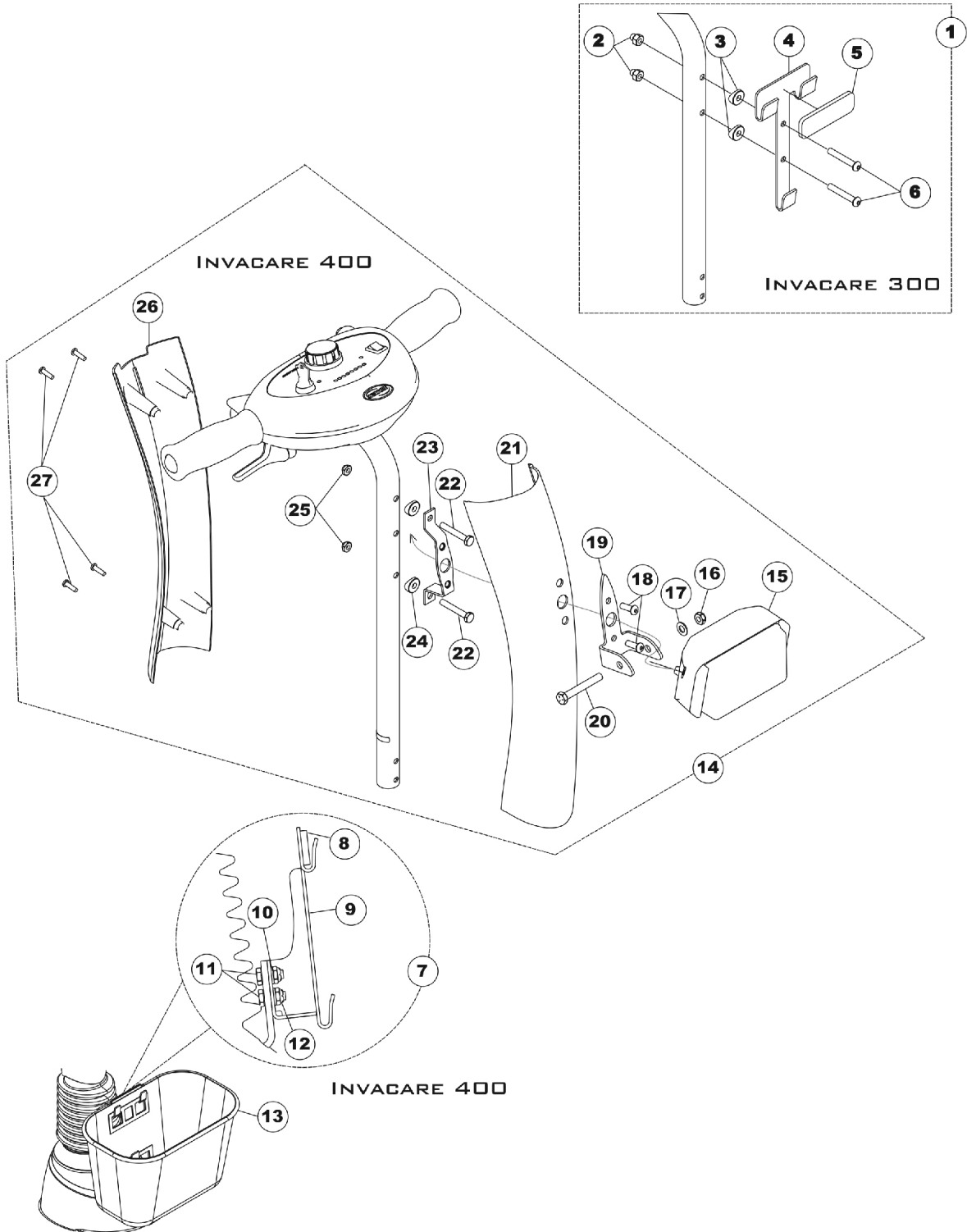

Basket, Headlight, and Tiller Shroud Invacare 400 Scooter

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1118432-NLA	NLA - Kit, Basket Mounting Hardware	1	1
2			Nut, Hex with Nylon Cap (1/4-20)	1	2
3			Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
4			NLA - Bracket, Basket Mounting, Black (Invacare 300 Scooter)	1	1
5			NLA - Pad, Foam	1	1
6			Screw, Socket Button Head (1/4-20 x 1-1/2")	1	2
7	B	1125364-NLA	NLA - Kit, Basket Mounting Hardware (Invacare 400 Scooter)	1	1
8			NLA - Pad, Foam	1	1
9			NLA - Bracket, Basket Mounting, Black (Invacare 400 Scooter)	1	1
10			NLA - Locknut (1/4-20)	1	2
11			Washer (1/4 x 1/2 x 1/16")	1	2
12			Screw, Hex Head (1/4-20 x 5/8")	1	2
13	C	1120484	Basket, Front	1	1
14	D	1125359-NLA	NLA - Tiller Shroud and Headlight Assembly - Candy Red Pearl	1	1
14	D	1125360-NLA	NLA - Tiller Shroud and Headlight Assembly - Deep Blue Pearl	1	1
15		1119187-NLA	NLA - Headlight	1	1
16		1028769	Locknut (5/16-18)	1	1
17		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	1
18		1091870	Screw, Socket Button Head w/ Patch (1/4-20 x 3/4")	1	2
19		1125357-NLA	NLA - Bracket, Light Mounting, Black	1	1
20		1070412-NLA	NLA - Screw, Hex Head (5/16-18 x 2-1/4")	1	1
21		1124888-NLA	NLA - Shroud, Front Tiller - Candy Red Pearl	1	1
21		1124889-NLA	NLA - Shroud, Front Tiller - Deep Blue Pearl	1	1
22		1037610	Screw, Hex Head (1/4-20 x 1-5/8")	1	2
23		1125358-NLA	NLA - Bracket, Tiller Shroud Mounting with Inserts, Black	1	1
24		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
25		1025195	NLA - Locknut (1/4-20)	1	2
		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
26		1114279-NLA	NLA - Shroud, Tiller, Black - Rear	1	1
27		1114276	Screw, Phillips Pan Head (#8-16 x 5/8")	1	4

NOTE: A - Includes: Items 2-6.
B - Includes: Items 8-12.
C - The Basket is the same for the HMV300 and the HMV 400.
D - Includes: Items 15-27.