


Back Cane Hardware w/o Back Brace

Fixed Height, Zero and Seven Degree

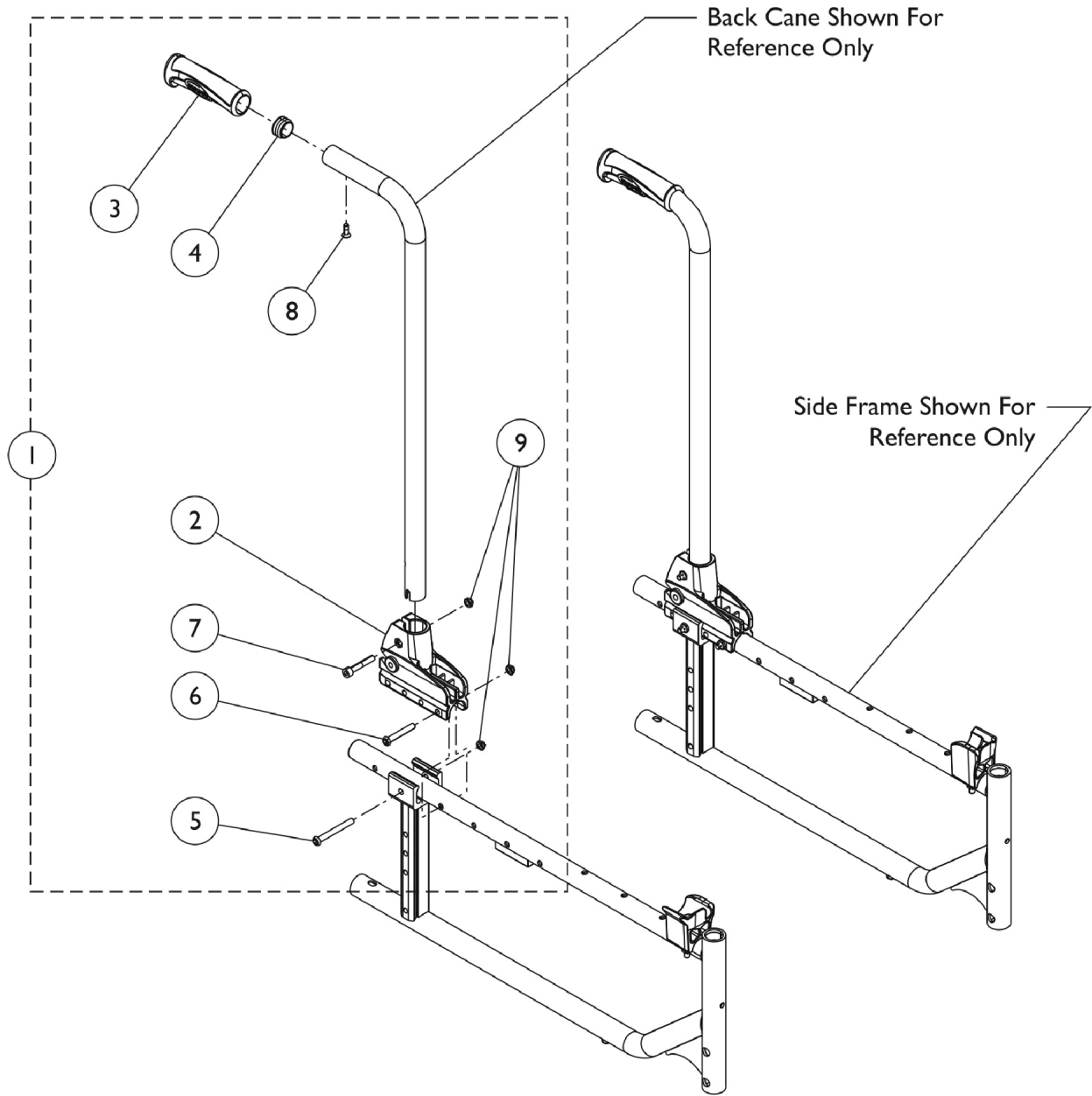

Back Cane Hardware w/o Back Brace Fixed Height, Zero and Seven Degree

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1169044-NLA	NLA - Hardware, Fixed Height Cane w/o Backbrace	1	1
2			Bracket, Cane, Rear	1	2
3			Grip, Cane (22 mm)	1	2
4			Button, Plug (22mm)	1	2
5			Screw, Socket Button Head (M6 x 53mm)	1	2
6			NLA - Screw, Socket Button Head (M6 x 40mm)	1	2
7			Screw, Socket Head, Cap (M6 x 35mm)	1	2
8			Screw, Phillips Pan Head (M4.8 X 16mm)	1	2
9			NLA - Locknut (M6 x 1)	1	6
2		1423410	Bracket, Cane, Rear	1	2

NOTE: A - Includes items 2 - 9.