


# Rear Shroud

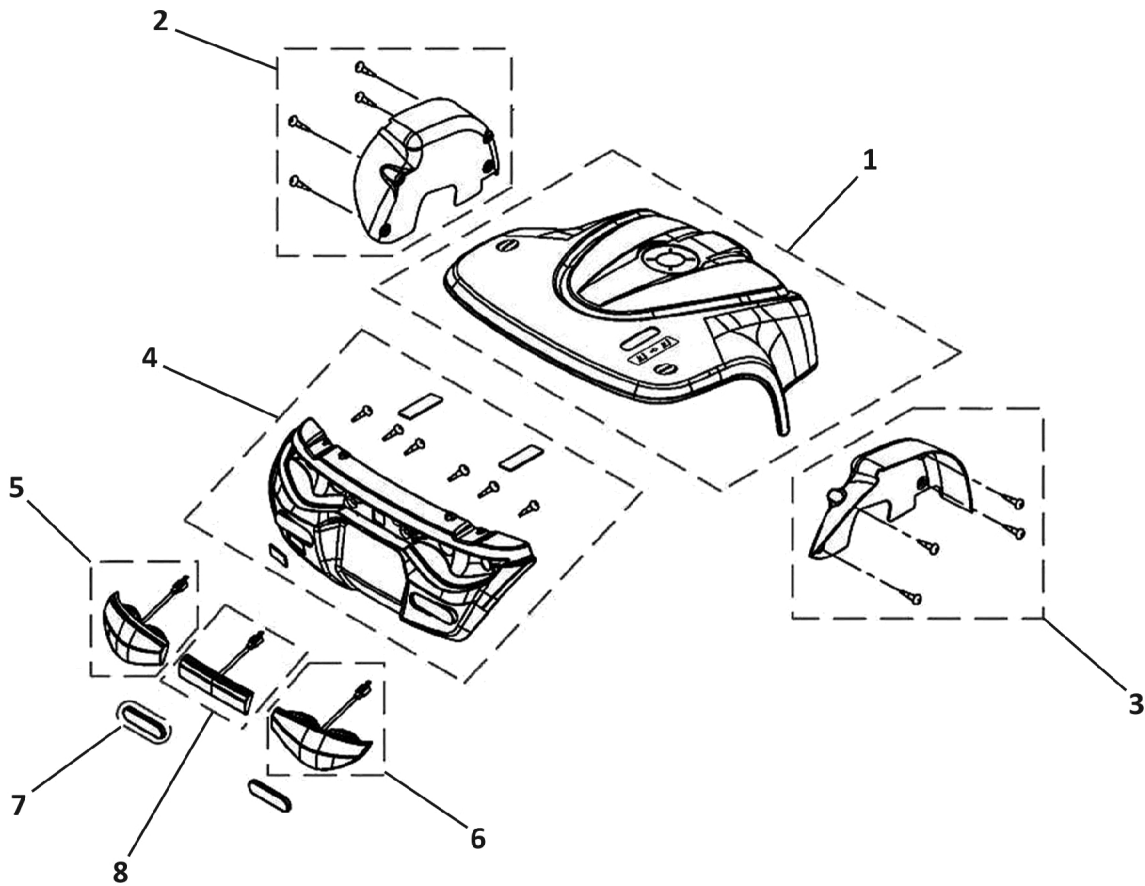

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1518037	Shroud, Rear, Complete, Silver	--	1
1		1521656	Shroud, Rear, Complete, Blue	--	1
1		1530003	Shroud, Rear, Complete, Red	--	1
2		1518030	Fender - Left	--	1
3		1518029	Fender - Right	--	1
4		1518032	Cover, Rear Bumper	--	1
5		1518034	Turn Signal, Rear - Left	--	1
6		1518033	Turn Signal, Rear - Right	--	1
7		1518036	Reflector, Rear, E-Mark	--	2
8		1518035	Neon Light, Complete	--	1