



Adjustable Angle Footplates with/without Pivot Tubes

(AT5543)

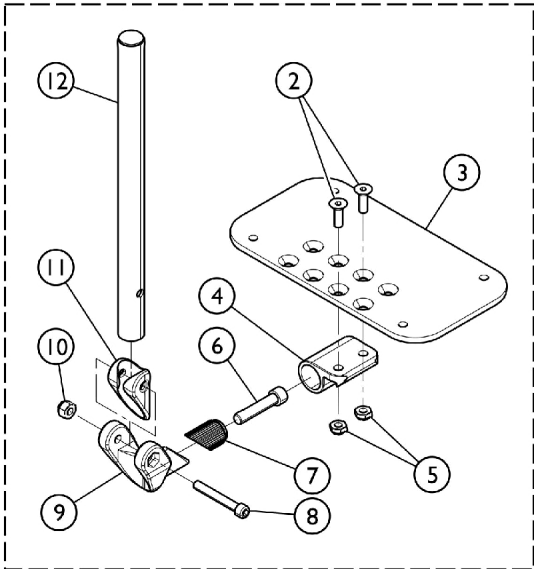


60 Degree, 60 Degree MFX, 70 Degree No Taper
70 Degree Lift-Off and 80 Degree No Taper
Swingaway, Articulating Legrests

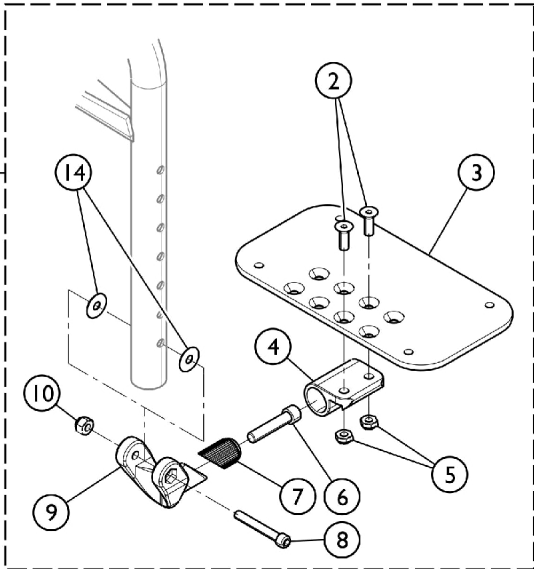

70 Degree MFX, 80 Degree MFX and
90 Degree Swingaway
Front Riggings

Used on All Front Riggings EXCEPT Wedge Style Elevating Legrests (AT5044/AT5044LIFT)

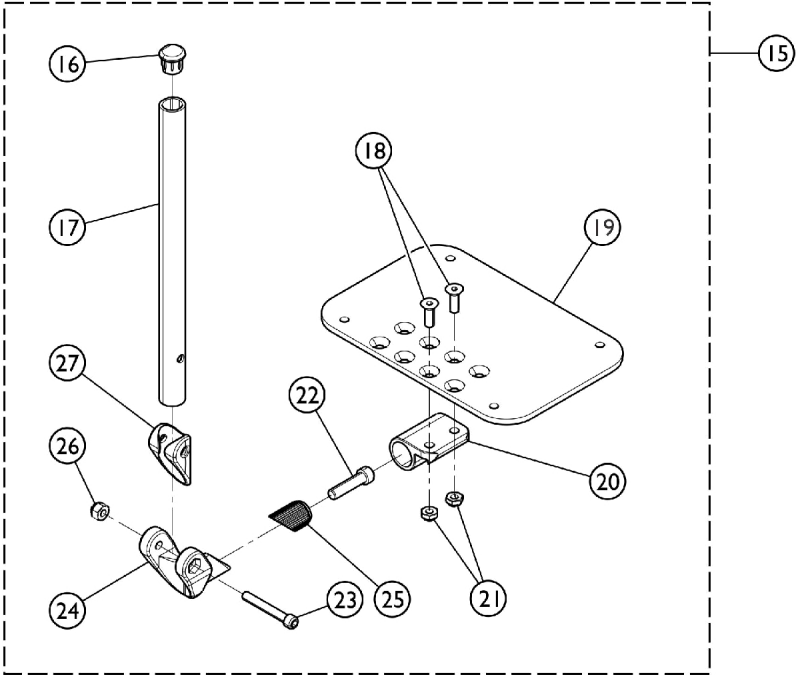

Used on Wedge Style Elevating Legrests (AT5044/AT5044LIFT)

Adjustable Angle Footplates with/without Pivot Tubes

(AT5543)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	See Chart	#1 60 Degree, 60 Degree MFX, 70 and 80 Degree Non Tapered, 70 Degree Lift Off Footrests	1	1
1	D	See Chart	#1 70 Degree Tapered Footrests	1	1
1	D	See Chart	#1 ATL4A Articulating Legrests	1	1
2		1113578	Screw, Socket Flat Head (1/4-28 x 3/4")	1	2
3		1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
3		1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
3		1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3		1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	1
4		1113528	Bracket, Adjustable Angle Footplate, Black	1	1
5		1025765	Locknut (1/4-28)	1	2
6		1021344	Screw, Socket Head (5/16-18 x 1-1/2")	1	1
7		1113530	Attachment, Hinge	1	1
8		1096810	Screw, Socket Head (1/4-28 x 1-3/4")	1	1
9		1113529	Hinge, Adjustable Angle Footrest	1	1
10		1065362	Locknut (1/4-28)	1	1
11		1114688	Spacer, Hinge, Black	1	1
12	A	1114699	Tube, Pivot/Slide, Straight, Black (10-1/4")	1	1
12	B	1114700	Tube, Pivot/Slide, Straight, 2" Longer ALPT, Black (12-1/4")	1	1
12	C	1114707	Tube, Pivot/Slide, Straight, Chrome (11-7/8")	1	1
13	E	See Chart	#13 70/80 Degree MFX, 90 Degree, and 70/90 Degree MFX Lift Off Footrests	1	1
14		1025456	Washer, Nylon (3/4 x 1/4 x 1/32")	1	2
15	F	See Chart	#15 Wedge Style AT5044 Elevating Legrests	1	1
16		1134427	Package, Plug Button, Black, "D" Shaped (3/4")	10	1
17	G	1134433	Pivot/Slide Tube w/ D Insert, Black	1	1
17	G	1134432	Pivot/Slide Tube w/ D Insert, Black (2" Longer)	1	1
18		1113578	Screw, Socket Flat Head (1/4-28 x 3/4")	1	2
19		1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
19		1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
19		1113575	Footplate, Adjustable Angle, XX- Large, Black (7-1/4" W x 8" L)	1	1
20		1113528	Bracket, Adjustable Angle Footplate, Black	1	1
21		1025765	Locknut (1/4-28)	1	2
22		1021343	Screw, Socket Head (5/16-18 x 1-1/4")	1	1
23		1096810	Screw, Socket Head (1/4-28 x 1-3/4")	1	1
24		1113529	Hinge, Adjustable Angle Footrest	1	1
25		1113530	Attachment, Hinge	1	1

Adjustable Angle Footplates with/without Pivot Tubes

(AT5543)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
26		1065362	Locknut (1/4-28)	1	1
27		1114688	Spacer, Hinge, Black	1	1
NOTE: A - 60 Degree, 60 Degree MFX, 70 Degree Tapered, 70 Degree Non Tapered, 70LIFTNT, 80 Degree Non Tapered Footrests B - ALPT 2" Longer C - ATL4A Articulating Legrests D - Includes items 2-12 E - Includes items 2-10 & 14 F - Includes items 16-27 G - Wedge Style AT5044 Legrest					

#1 Adjustable Angle Footplates w/ Pivot Tubes

SWING1319, SWING914, 70NTAPER, 70LIFTNT, and 80NTAPER Footrests

Straight Slide Tube (10-1/4")	Right	Left	ALPT Straight Slide Tube (12-1/4")	Right	Left
12" - 13" Wide (4-1/4" Footplate)	1114640	1114641	12" - 13" Wide (4-1/4" Footplate)	1114650	1114651
14" - 15" Wide (5-1/4" Footplate)	1114642	1114643	14" - 15" Wide (5-1/4" Footplate)	1114652	1114653
16" - 17" Wide (6-1/4" Footplate)	1114644	1114645	16" - 17" Wide (6-1/4" Footplate)	1114654	1114655
18" - 24" Wide (7-1/4" Footplate)	1114646	1114647	18" - 24" Wide (7-1/4" Footplate)	1114656	1114657

#1 Adjustable Angle Footplates w/ Pivot/Slide Tubes

70STAPER Footrests

Straight Slide Tube (10-1/4")	Right	Left	ALPT Straight Slide Tube (12-1/4")	Right	Left
16" Wide (4-1/4" Footplate)	1114640	1114641	16" Wide (4-1/4" Footplate)	1114650	1114651
17" - 18" Wide (5-1/4" Footplate)	1114642	1114643	17" - 18" Wide (5-1/4" Footplate)	1114652	1114653
19" - 24" Wide (6-1/4" Footplate)	1114644	1114645	19" - 24" Wide (6-1/4" Footplate)	1114654	1114655

#1 Adjustable Angle Footplates w/ Pivot Tubes

ATL4A Articulating Legrests

Adjustable Angle Footplates with/without Pivot Tubes

(AT5543)

#1 Adjustable Angle Footplates w/ Pivot Tubes

ATL4A Articulating Legrests

Straight Slide Tube (10-1/4")	Right	Left
14" - 15" Wide (5-1/4" Footplate)	1114708	1114709
16" - 17" Wide (6-1/4" Footplate)	1114710	1114711
18" - 24" Wide (7-1/4" Footplate)	1114712	1114713

Adjustable Angle Footplates with/without Pivot Tubes

(AT5543)

#13 Adjustable Angle Footplates w/o Pivot Tubes

70MFX, 80MFX, 90SWING, 70LIFTMX, and 90LIFTMX Footrests

No Slide Tube - Bolts Through Support	Right	Left
12" - 13" Wide (4-1/4" Footplate)	1114630	1114631
14" - 15" Wide (5-1/4" Footplate)	1114632	1114633
16" - 17" Wide (6-1/4" Footplate)	1114634	1114635
18" - 24" Wide (7-1/4" Footplate)	1114636	1114637

#15 Adjustable Angle Footplates w/ Pivot Tubes

Wedge Style AT5044 Elevating Legrests

Straight Slide Tube (10-1/4")	Right	Left	ALPT Straight Slide Tube (12-1/4")	Right	Left
14" - 15" Wide (5-1/4" Footplate)	1132061	1132062	14" - 15" Wide (5-1/4" Footplate)	1132089	1132090
16" - 17" Wide (6-1/4" Footplate)	1132063	1132064	16" - 17" Wide (6-1/4" Footplate)	1132091	1132092
18" - 24" Wide (7-1/4" Footplate)	1132065	1132066	18" - 24" Wide (7-1/4" Footplate)	1132093	1132094