


# Wheels - Rear - Service Parts - Composite (COM)

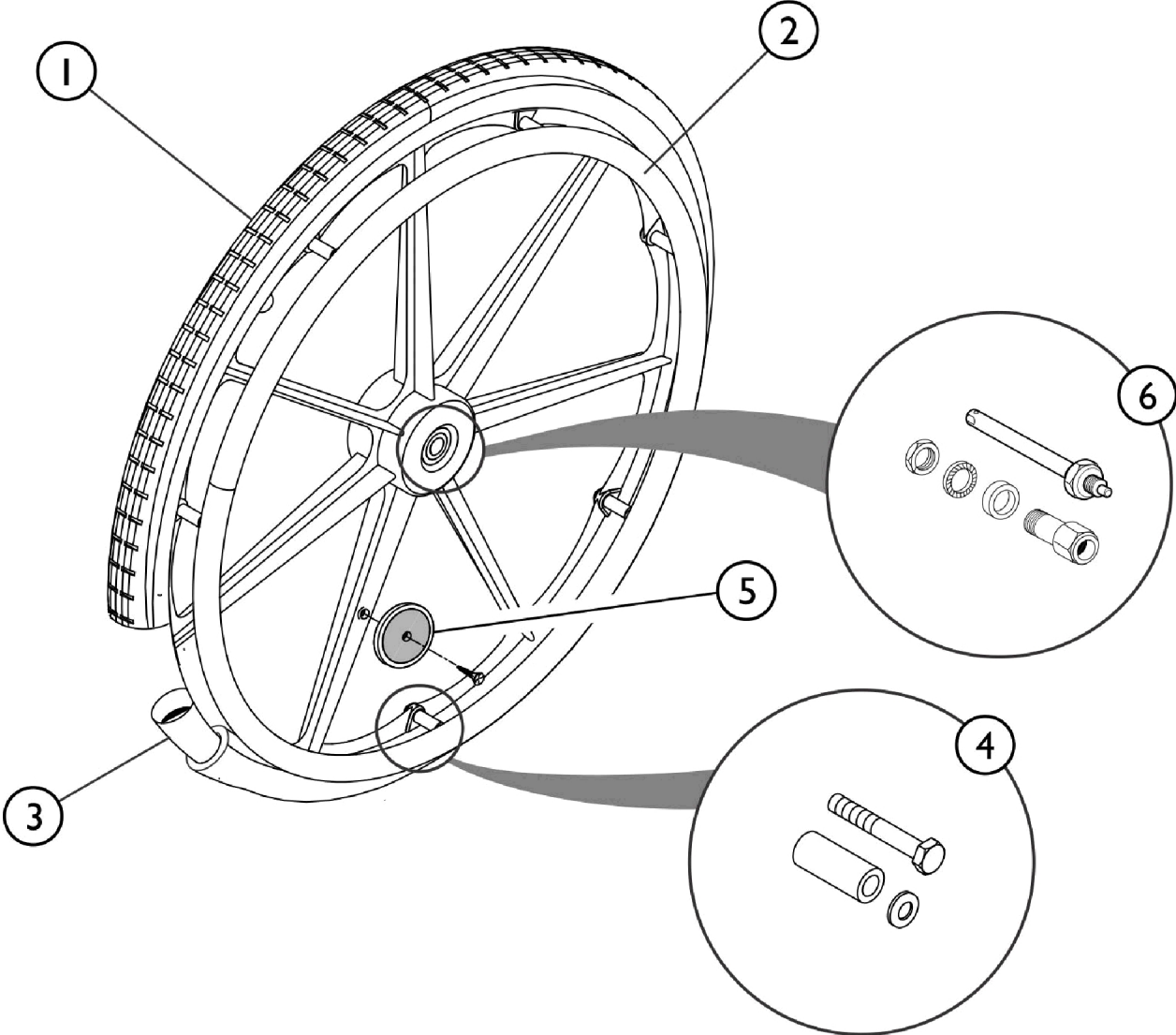

SCM\_121A

## Wheels - Rear - Service Parts - Composite (COM)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		SP0036-30035	Tire, Pneumatic, Grey (22" x 1-3/8")	1	1
1		SP0036-10014	Tire, Solid, Grey (22" x 1-3/8")	1	1
1		SP0036-30001	Tire, Pneumatic, Grey (24" x 1-3/8")	1	1
1		SP0036-10013	Tire, Solid, Grey (24" x 1-3/8")	1	1
2		SP1604839	Kit, Aluminum Handrim, 7 Position, Composite Wheel (22")	1	1
2		SP1601840	Kit, Anti-Slip Handrim, 7 Position, Composite Wheel (22")	1	1
2		SP1601841	Kit, Aluminum Handrim , 7 Position, Composite Wheel (24")	1	1
2		SP1601842	Kit, Anti-Slip Handrim, 7 Position, Composite Wheel (24")	1	1
3		SP0036-20007	Tube, Inner (22" x 1-3/8")	1	1
3		SP0036-20001	Tube, Inner (24" x 1-3/8")	1	1
4		SP1601484	Kit, Handrim Hardware, Composite Wheel, 7 position	1	1
5		SP9037-30001	Kit, Reflector, Composite Wheel	1	1
6		SP1591280	Kit, Quick Release Axle	1	1