


# Legrest Support Assembly

## 92 - (12', 14", 16" Wide Chairs)

Calf Pads are not included with support assembly. Please refer to "Front Rigging Index and Calf Pads" category.

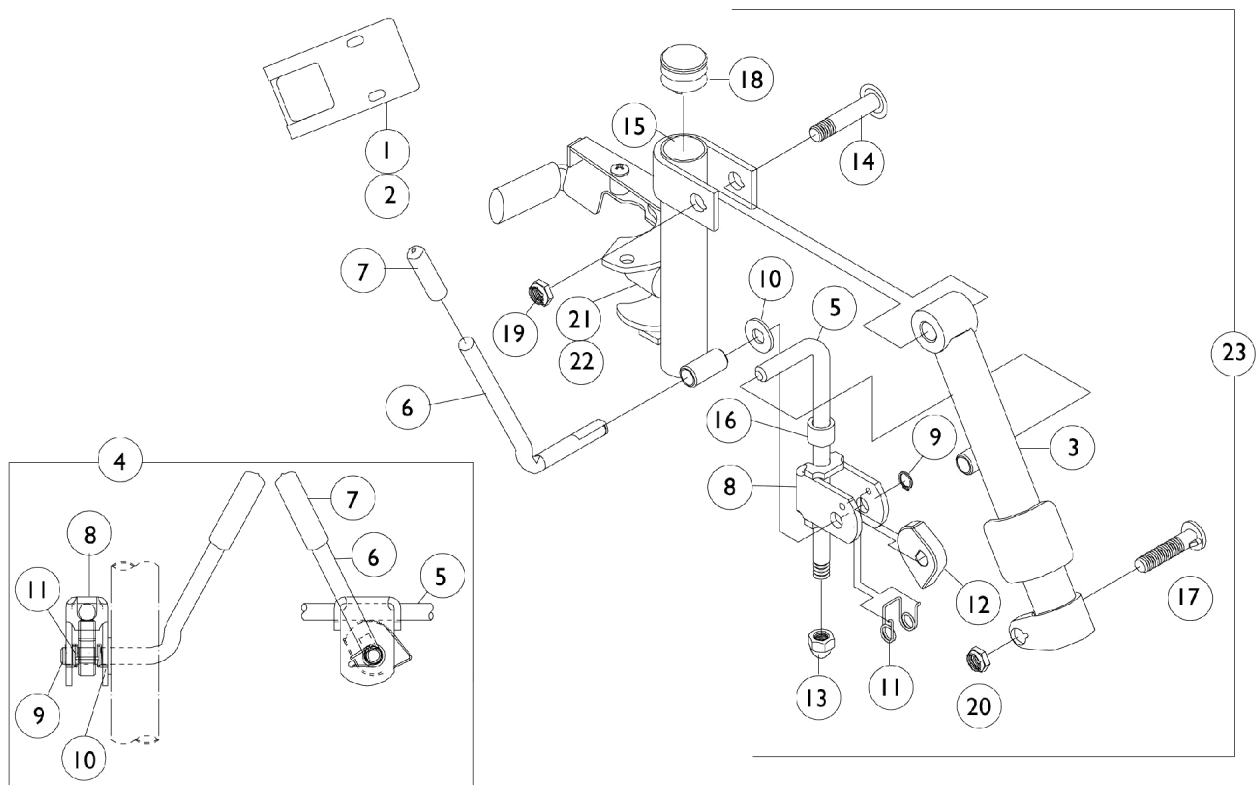

# Legrest Support Assembly

## 92 - (12', 14", 16" Wide Chairs)

Calf Pads are not included with support assembly. Please refer to "Front Riggings Index and Calf Pads" category.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		47352X003	Lock, Pad Plate, Legrest, Black	1	1
2		47352X004	Clamp, Black	1	1
3	F	04526X006-NLA	NLA - Tube, Elevating Support, Chrome - Right	1	1
3	F	04526X007-NLA	NLA - Tube, Elevating Support, Chrome - Left	1	1
3	G	04526X002-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Right	1	1
3	G	04526X003-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Left	1	1
3	H	04526X004-NLA	NLA - Tube, Elevating Support, Chrome - Right	1	1
3	H	04526X005-NLA	NLA - Tube, Elevating Support, Chrome - Left	1	1
4	A,E	1046328-NLA	NLA - Kit, Ratchet	1	1
5		42326X001	Bar, Ratchet	1	1
6	A	1037126	Handle, Release	1	1
7	C	1040060	Tip, Plastic, Black	1	1
7	D	1006490	Tip, Plastic, Black	1	1
8	A	04528X002	Bracket, Release, Black	1	1
9	B,C	1040831	Ring, Snap	1	1
9	B,D	40187X000-NLA	NLA - Pin, Roll (1/8" x 9/16")	1	1
10		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
11	A	1037485	Spring, Release	1	1
12	A	1037125	Cam, Legrest	1	1
13		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
14		11919X000-NLA	NLA - Screw, Slotted Truss Head (3/8-16 x 1-1/2")	1	1
15		1031481-NLA	NLA - Legrest Support Tube Assembly, Adult, Chrome - Right	1	1
15		1031482-NLA	NLA - Legrest Support Tube Assembly, Adult, Chrome - Left	1	1
16		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
17		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
18		1027095	Plug Button, Black (7/8")	1	1
19		40363X000	Hex Nut (3/8"-16)	1	1
20		1107321	Nut, Low Cap (5/16"-24)	1	1
21		1022874	Bumper	1	1
22		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
23	C,F	1031485-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Right	1	1
23	C,F	1031486-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Left	1	1
23	D,G	1031487-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Right	1	1
23	D,G	1031488-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Left	1	1
23	H	1031483-NLA	NLA - Legrest Support Assembly, Adult, Chrome - Right	1	1
23	H	1031484	Legrest Support Assembly, Adult, Chrome - Left	1	1
	I	See Chart	Pivot Slide Tube with Aluminum Footplate	1	1

# Legrest Support Assembly

## 92 - (12', 14", 16" Wide Chairs)

**Calf Pads are not included with support assembly. Please refer to "Front Riggings Index and Calf Pads" category.**

NOTE: A - Due to product improvement - items 6, 8, 11, and 12 are NLA for chairs built before 6/94. Ratchet Bar Kit (item 4) must be ordered for those chairs.  
 B - After 5/94, the snap ring replaced the roll pin to attach the ratchet handle to the ratchet.  
 C - After 5/94  
 D - Before 6/94  
 E - Includes items 5-13  
 F - 12" Wide Chairs  
 G - 14" Wide Chairs  
 H - 16" Wide Chairs  
 I - Not Shown

### Pivot Slide Tube with Aluminum Footplate

Chair Width	Standard 9-1/2"	2" Longer 11-1/2"	4" Longer 13-1/2"
12"	1007280-NLA See Note(s) A	N/A	N/A
	1007281-NLA See Note(s) B	N/A	N/A
14"	1007271-NLA See Note(s) A	N/A	N/A
	1007272-NLA See Note(s) B	N/A	N/A
16"	1017237-NLA See Note(s) A	1086525-NLA See Note(s) A	1086526-NLA See Note(s) A
	1017238-NLA See Note(s) B	1086527-NLA See Note(s) B	1086528-NLA See Note(s) B

NOTE: A - Right  
 B - Left

The following pivot slide tubes with footplates are used with the above assembly. For more information on the pivot slide tube or footplates, see page titled "Pivot Slide Tube with Footplate."